

# INSTRUCTIONS FOR USE

- 1) All items to be sterilized should be thoroughly cleaned in accordance with standard procedures.
- 2) Affix pressure sensitive label to each item to be sterilized.
- 3) Sterilize item as usual in Dry Heat Sterilizer.
- 4) Chemical indicator bar turns from green to dark brown/black in approximately 10 minutes at 160°C.
- 5) If indicator bar has not completed color change from green to dark brown/black, investigate as a cycle failure.

Mesa Dry Heat Indicator Labels should be stored at room temperature. Protect from moisture and excessive humidity. Labels are valid for three years beyond the date of manufacture. The first four digits of the Lot number indicate the year and month of manufacture (YY/MM).

*Manufactured For:*

**Mesa Labs**

625 Zoot Way

Bozeman, MT 59718

303-987-8000 or 1-800-992-6372

bisupport@mesalabs.com

[www.MesaLabs.com](http://www.MesaLabs.com)



**MesaLabs**

**DHI-INSL Rev 4.0**

LB-013 Rev. H